

ACE® CUVETTES

RINSING, PACKING AND SHIPPING PROCEDURES

Items Supplied

- 1 – Shipping Carton With Cover Tray
- 1 – Rinsing Container with Solid Lid
- 1 – Drain Lid
- 4 – Bottles Sodium Hypochlorite (Bleach)
- 1 – Shipping Canister with Yellow Lid
- 1 – FedEx Ground Return Record

PROTECTIVE GLOVES SHOULD BE WORN DURING RINSING PROCESS

1. LOAD

At end of shift empty used cuvettes from the instrument waste tray into the white Rinsing Container provided.

2. FILL

(Loaded container needs to be taken to a convenient rinsing station.)
Empty ½ bottle of Sodium Hypochlorite (Bleach) into the rinsing container and fill with D.I. water as indicated.

3. RINSE

Seal container with solid lid. Hold over the sink and invert 3 or 4 times so that all cuvettes are exposed to the liquid and let stand for 5-10 minutes.

4. DRAIN

Replace Solid Lid with Drain Lid and pour solution down drain. Inverting the container a second and third time will allow for additional liquid to be shed from cuvettes.

5. BAG


Empty decontaminated cuvettes into Shipping Canister as indicated. Place Yellow Lid on Container and store in Shipping Carton. Repeat until approximately 140 Sleeves worth of Cuvettes have been used.

6. BOX


Place Shipping Canister with Decontaminated Cuvettes into Shipping Carton as shown. Enclose empty bleach bottles, used Cuvette Sleeves and the completed FedEx shipping record.

7. SHIP




Seal shipping carton with preprinted return label appearing on outside flap. Call FedEx Ground at **1-800-937-4587** for pick up.

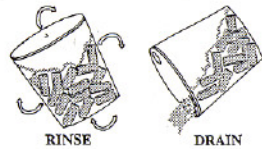

DRAIN

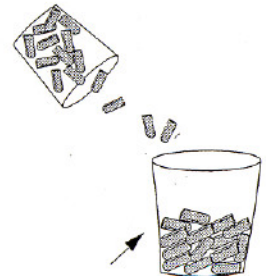

Cuvette Sleeves

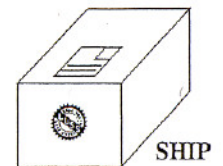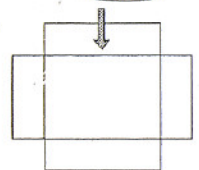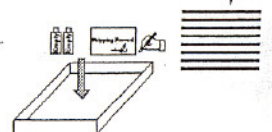

NOTE: PLEASE RETAIN RINSING CONTAINER AND LIDS FOR FUTURE USE.

ACE® is a Registered Trademark of Alfa Wassermann