

# COULTER/I.L. ACL® ROTORS

## RINSING PROCEDURES

For Our Standard Volume Users

### Items Supplied

- 4 – Rinsing Canisters
- 2-Bottles of Decontamination Solution
- 1 – Kit (Bags, Rods & Spacers)
- 1 –FedEx Return Record

### 1. PREPARE

Fill canister ½ full with DI water and ½ bottle decontamination solution. Insert alignment rod with ring on bottom of rod.

### 2. LOAD

After completion of tests, stack rotors onto rod with wells facing up. A spacer should be placed between each rotor. The canister will hold 18 rotors. Do not over pack canister.

### 3. RINSE

Rotate rod in a circular motion, secure solid lid, and rotate canister 8 – 10 times. Allow rotors to soak for 6 – 24 hours.

### 4. DRAIN

After soaking is completed, place drain lid on canister and drain liquid thoroughly.

### 5. BOX

Replace drain lid with solid lid, close tightly, and put in box. Box will hold 4 canisters. (72 rotors)

### 6. SHIP

Once box is full, enclose a completed FedEx Ground pickup record and empty solution bottles. Seal box with preprinted address on outside flap and call FedEx at 1-800-937-4587 for pickup.

