

# Bio Chem ATAC 6000 Cuvette Strips®

**THE KEY TO SUCCESSFUL CUVETTE REPROCESSING IS IMMEDIATE AND THOROUGH RINSING OF THE USED CUVETTES**

## FOR OUR LOW VOLUME USERS

1. **LOAD**  
Place used cuvette strips into container after each use.
2. **RINSE**  
Fill container covering strips with DI water and add 1/2 bottle decontaminating solution.
3. **ROTATE**  
Secure lid, swirl container 4 - 5 times ensuring complete coverage.
4. **DRAIN**  
Replace top with drain lid and thoroughly drain by shaking container as solution is being dispensed.
5. **BAG**  
Transfer decontaminated cuvette strips to bag, twist tie close and place in shipping carton. (Box will hold 100 strips - 2 bags of 50).
6. **BOX**  
Enclose completed packing slip and empty solution bottles.
7. **SHIP**  
Seal carton with pre-affixed return label on outside flap.

