

# Bayer/Technicon RA-1000® TRAYS

## RINSING PROCEDURES

### Items Supplied

- 1- Rinsing Canister (retain for future use)
- 4 - Bottles of Decontamination Solution
- 2 - Plastic Bags
- 1 - FedEx Return Record

#### 1. PREPARE

After completion of tests, stack trays in canister with wells facing up. The canister will hold 4 trays.

#### 2. RINSE

Fill canister 3 liters full with DI water and add 1/3 bottle decontamination solution. Secure lid and invert canister 4-5 times. Trays should not soak more than 8 hours.

#### 3. DRAIN

Replace lid with drain lid and thoroughly drain liquid. Do not allow trays to dry. Bag trays directly after draining.

#### 4. BAG & BOX

Transfer trays to bag, twist tie close and place in box. The box will hold two bags of 10 trays. (20 trays per box.) Place plastic insert on top of bags to secure trays during shipping.

#### 5. SHIP

Once box is full, enclose a completed FedEx pickup record and empty solution bottles. Seal box with preprinted address on outside flap and call FedEx at 1-800-937-4587 for pickup.

