

# BIOMERIEUX/ORGANON TEKNIKA RA-4® TRAYS

## RINSING PROCEDURES

### Items Supplied

- 4 – Rinsing Canisters (Retain for Future Use)
- 4 – Bottles of Decontamination Solution
- 1 – Kit of Spindles & Spacers
- 1 – Box Liner
- 1 – FedEx Pickup Record

#### 1. PREPARE

Place gray spindle and spacers next to instrument. After testing, stack trays on spindle with wells facing up. Insert spacers between each tray on spindle. The spindle will hold 14 trays.

#### 2. RINSE

Rinse when either spindle is full or the day's testing is complete. Fill canister ½ full with DI water and ½ bottle of decontamination solution. Submerge spindle into canister. Secure lid and rotate canister 8-10 times.

#### 3. DRAIN

Replace solid lid with drain lid and thoroughly drain liquid from canister.

#### 4. BOX

Replace drain lid with solid lid, close tightly, and put in box liner. The box will hold 4 canisters of 14 trays. (56 trays)

#### 5. SHIP

Once box is full, enclose a completed FedEx pickup record and empty solution bottles. Seal box with preprinted return address on outside flap and call FedEx at 1-800-937-4587 for pickup.

